

**NEW**



**EXPERIENCE THE DIFFERENCE.**

**CONCEPT**<sup>TM</sup>  
*suture passer*

# The CONCEPT is... Suture Passing SIMPLIFIED.



**Insert Needle:** Align tab with laser mark.



**Load Suture:** Suture can be loaded in any direction at and location on the suture.



**Engage Tissue:** Squeeze the Jaw Lever to grasp tissue.



**Fire Needle:** Keep Jaw lever engaged and squeeze the needle trigger to fire the suture through tissue.



**Retrieve Suture:** Release Needle Trigger. Keep Jaw Lever engaged and the needle will retract as the active window capture the suture. Release jaw lever.

**Remove Suture:** Place slight pressure on the needle trigger and remove suture. Use Caution: if you fully engage the needle trigger the needle will fire.



Visit [SRS.Linvatec.com](http://SRS.Linvatec.com) today to learn more.



SHOULDER RESTORATION SYSTEM

11311 Concept Boulevard • Largo, Florida 33773-4908 • Customer Service (800) 237-0169  
USA Fax (727) 399-5256 • International Fax (727) 397-4540 • [www.linvatec.com](http://www.linvatec.com)  
©2010 Linvatec Corporation, a Subsidiary of ConMed Corporation CST 3037

