

*Surgical Innovation | Customer Driven*

### STRONG PEEK CF


- Radiolucent and MR safe
- Mechanical properties close to cortical bone

### LARGE EYELET

- Can accommodate up to 6 suture tails

### DESIGN

- Single piece for simplified insertion
- Drive-in fixation
- Choice of reusable or disposable driver

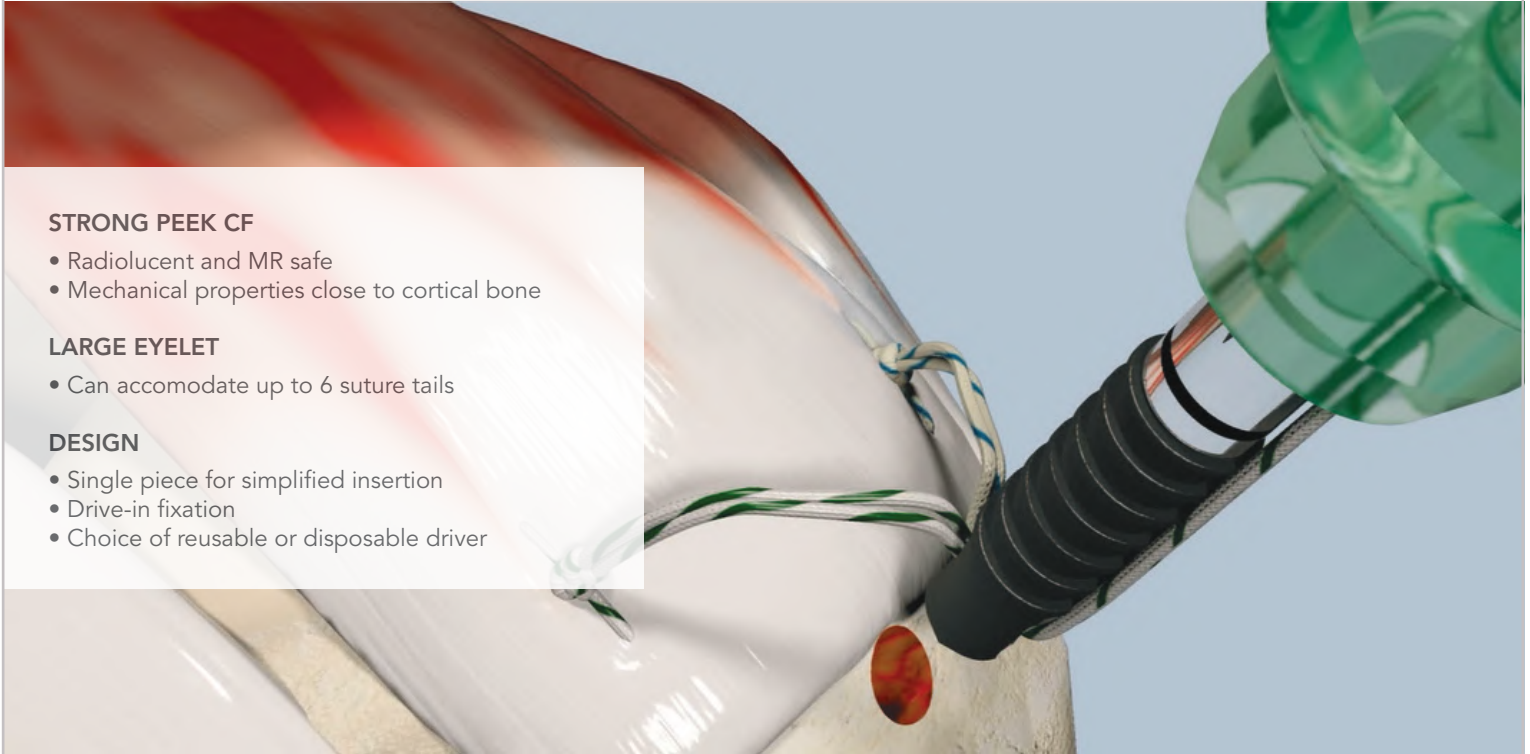

## ORDERING INFORMATION

Knotless PEEK CF Push-In Suture Anchors	
Part #	Description
10314	45 Knotless PEEK CF Push-In Suture Anchor w/Suture Passer
10315	55 Knotless PEEK CF Push-In Suture Anchor w/Suture Passer

Required Instrumentation for Knotless PEEK CF Push-In Suture Anchors	
Part #	Description
10345	45 or 55 Knotless Suture Anchor Driver (Use w/10314 or 10315 Suture Anchor)
10487	45 or 55 Knotless Suture Anchor Driver, Unsterile Reusable (Use w/10314 or 10315 Suture Anchor)
10365	3.2mm Awl w/Handle

Additional sizes and instruments available.



*Surgical Innovation | Customer Driven*



step **1** AWL to first line.



step **2** Load all sutures through the Knotless Anchor using the supplies suture passer.




step **3** Firmly place tip of Knotless in pilot hole.



step **4** Remove slack in the sutures.



step **5** Tap Knotless Anchor - final tension achieved on insertion.

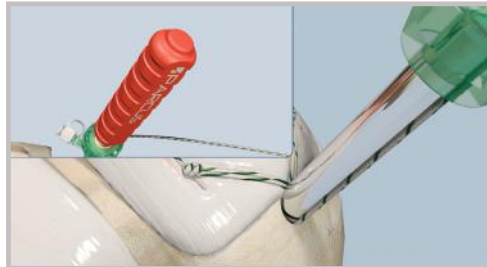

step **6** Disengage driver by rotating and pulling back.



step **7** Shown with Draw Tight 3.2mm suture anchor placed medially.



step **8** Completed repair.